NEOPT® multi

Lubrifiant oculaire

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Eye lubricant

INSTRUCTIONS FOR USE**EN****Description:**

NEOPT® multi is a preserved sterile hypotonic eye lubricant, containing sodium hyaluronate at 0.2%.

NEOPT® multi maintains and restores the physiological conditions of the eye surface, protects its cells, induces a long lasting comfort.

Indications:

Dry eye symptoms.

NEOPT® multi brings prompt and lasting relief in case of sensation of dryness, irritated and tired eyes caused by dry eye disease and by external factors such as prolonged exposure to video screens, air conditioning, dust, pollution, smoke, etc.

NEOPT® multi can be used by wearers of all types of corrective contact lenses.

Composition:

Sodium Hyaluronate 0.2 %

Sodium chloride, boric acid, borax, dequest solution, sodium perborate tetrahydrate and purified water.

Presentation:

15 ml multidose ophthalmic container.

Directions for use:

- Wash and rinse your hands thoroughly.
- Remove the protective cap. Make sure that the tip of the bottle does not touch your fingers.
- Lean your head back slightly and pull your lower eyelid down slightly with a finger of your free hand and hold the bottle vertically above your eye with the nozzle pointing downwards. Then gently squeeze the bottle to apply one or two drops into each eye, or according to the specialist's advice. Avoid contact of the top of the bottle with eye or your fingers to prevent any possible contamination of the solution.
- Replace the protective cap after use.
- The product can be used whenever necessary

Precautions:

- Do not use if the bottle is damaged (risk of microbial contamination).
- Do not share the bottle to prevent cross contamination.
- Close the bottle carefully after each use (risk of microbial contamination).

Warnings:

- If your doctor has prescribed other eye drops, observe an interval of at least 15 minutes between the two instillations.
- To prevent possible contamination, do not let the tip of the dropper come into contact with any surface.
- Do not use if you are hypersensitive (allergic) to any of the constituents.
- Rare cases of mild eye irritation are possible after product instillation. Stop using it if you experience pain, redness or any change in vision following the instillation. Ask a doctor for advice if the symptoms persist or get worse.
- If you experience blurred vision after application, wait until your vision is clear before driving or using tools or machinery.
- Do not swallow the solution.
- Keep out of the reach and sight of children.
- If you experience a persistent problem, please consult your doctor/ophthalmologist.
- Do not instill NEOPT® multi in the eye(s) together with other substances as their effect may be altered.

Storage Conditions:

- Store the bottle at a temperature below 30°C.
- Do not use the solution after the expiry date.
- After the bottle has been opened, the solution must be used within 90 days. Any residual product must be disposed of.

Medical device compliant with the requirements of Directive 93/42/EEC amended

Date CE marking affixed: february 2015

Date of last revision of the text: february 2015

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Tubilux Pharma SpA- Via Costarica, 20/22 - 00071 Pomezia (RM) Italy